



CLEANING INSTRUCTIONS

General:

■ **Metal parts (RAL 7035 / RAL 9005 / RAL 9006 or chrome)**

- Apply bleach with 9.6% active chlorine, Vigor-type detergent diluted to 5% or ANIOS-type products (finish tests on painted metal by the FCBA (French Institute of Technology for Forest-based and Furniture Sectors) in 2007 in accordance with technical requirements 25.SC.01 and 22.NFS.01)
- Clean using a single-use cloth
- Rinse thoroughly
- Dry thoroughly after cleaning

■ **Plastic parts**

- Wash and clean using a damp single-use cloth.

■ **Coverings**

- Use warm soapy water or ANIOS-type disinfectant (a set of ANIOS products was tested on coated coverings and they do not change the quality of the covering: ANIOS DDSH, ANIOS TSA, SURFANIOS and ANIOSURF).
- Clean using a single-use cloth, taking particular care with the following parts of the seat: headrest, seat, and more particularly the front of the seat in contact with the legs. These are the areas that are most often exposed to the phenomenon of deplastification of coated covering in the event of careless cleaning.
- Rinse and dry.

Steam disinfection is compatible with all Navailles coverings.

List of products that must in no circumstances be used:

- Undiluted alcohol and/or acetone
- All solvents and detergents
- Abrasive cleaning products
- Perchloroethylene
- All types of waxes
- Trichloroethylene

Special cases:

■ Ink, persistent stains:

Whether cleaning with soapy water, dilute alcohol (30% alcohol, 70% water) or dilute acetone (10% acetone, 90% water), ball point ink-type stains can be made less obvious but cannot be removed. Rinse with clean water.

■ Eosin and medical products

Wash with soapy water as soon as possible. For persistent stains, use bleach or a 10% dilution of alcohol, but the stain is likely to persist. Rinse with clean water.

The dyes in some medical products migrate into materials and leave an indelible stain that will fade with cleaning against dark and printed colours.

Hospital disinfectants from the ANIOS range do not stain coverings.

■ Food, ketchup, chocolate

Stains can be removed by washing with soapy water. A stain ring may persist depending on the colour of the fabric (especially if it is light and single coloured).

Rapid cleaning of the stain is advised.

For persistent stains, use bleach or a 10% dilution of alcohol, which will reduce the stain ring. Rinse with clean water.



Here you can find our Ravello models: www.mc-seniorenprodukte.de